



# ERGO INSTRUCTION MANUAL



**WIRELESS REMOTE BUTTON**

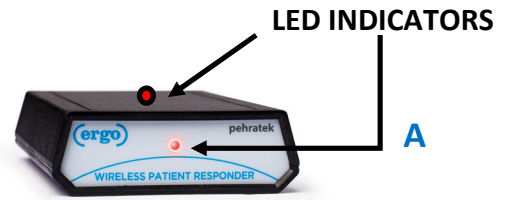

**WIRELESS SIGNALLING RECEIVER**

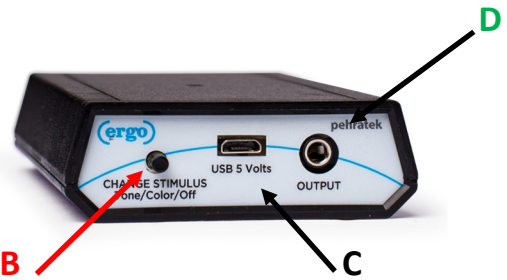

Recharge Remote 4 Hours before using  
(Use receiver power USB cable)  
LED –Yellow when charging  
LED-Green when fully charged

**PATIENT INSTRUCTIONS:**


Press blue dot to respond with finger or thumb from either hand  
  
The remote will flash the LED red and gently vibrate confirming their response.

**CLEANING -**

Wipe down remote after each use with an antibacterial wipes  
(DO NOT USE ALCOHOL ON UNIT)

**TO CHANGE CHANNELS**

Reposition DIP switches to be of the same pattern on the remote and receiver.

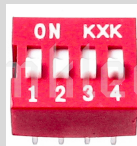

Default is All "ON"

**A**-Position near audiometer for best visibility and or audibility to the examiner

**C** Plug in USB power adapter to power up unit

**B** Select desired signaling mode by pressing button for 2 seconds to cycle through options. Press again to test selection.

- Red LED-No sound**
- Red Flashing-No sound**
- Green LED-No sound**
- Red LED –Low sound**
- Red LED-High sound**

**D** OPTIONAL-Connect output to audiometer to utilize built in patient response light (adapter included )

PEHRATEK PRODUCTS  
9285 Pierson Lake Drive  
Chaska, MN 55318  
(952) 470-9063

|              |             |
|--------------|-------------|
| WHITE        | Standby     |
| BLUE         | ACTIVATED   |
| RED FLASHING | LOW BATT    |
| YELLOW       | CHARGING    |
| GREEN        | FULL CHARGE |